



# Kompasu View MD3 Alpha

**Test 3 items simultaneous !**

**Easy, quick and no measuring equipment required**

- Blood glucose
  - Cholesterol
  - Triglyceride
- Fingertip blood 3 minutes**

With a small amount of fingertip blood, KOMPASU View MD3 Alpha can measure 3 items (blood glucose, triglyceride and cholesterol) simultaneously in just 3

## KOMPASU View MD3 Alpha

KOMPASU View MD3 Alpha is a vertical flow reagent paper device for the detection of blood glucose, triglyceride and cholesterol in whole blood.

### Clinical performance measurement

#### Blood glucose

Total participants	50
Match number	44
Coincident ratio [%]	88

※ Level 1 : under 80 [mg/dL], Level 2 : 81 - 125 [mg/dL], Level 3 : over 126 [mg/dL]

#### Triglyceride

Total participants	50
Match number	48
Coincident ratio [%]	96

※ Level 1 : under 150 [mg/dL], Level 2 : 151 - 250 [mg/dL], Level 3 : over 251 [mg/dL]

#### Cholesterol

Total participants	50
Match number	45
Coincident ratio [%]	90

※ Level 1 : under 140 [mg/dL], Level 2 : 141 - 250 [mg/dL], Level 3 : over 251 [mg/dL]

※ The capable of showing an excessive, sufficient, deficient level of blood-glucose, triglyceride and cholesterol in whole blood.

※ Performance data obtained by clinical study with 50 participants enrolled.

## Product information

Product name	KOMPASU View MD3 Alpha
Sample	Whole Blood
Format	Reagent paper device
Parameter	Blood glucose, Triglyceride and Cholesterol
Time to result	3 [min]
Storage condition	2-25 [°C]
Test principle	Colorimetric Analysis

## How to use it

- 1 Clench and relax the fist  
Improve the blood circulation
- 2 Prick the fingertip
- 3 Take a blood sample after pricking the finger. Allow the blood to soak into the test device.
- 4 Fold the sample holder and wait 3minutes before reading the results.

Align the slider with the standard color chart to read the results.